



Mouse anti- Vimentin

Cat. No.: AIB-30182 (1 ml Concentrate); AIB-30183 (0.5 ml Concentrate); AIB-30181 (6 ml Ready-to-use)

Instructions for use

Intended use

This antibody is designed for the specific localisation of Vimentin in formalin-fixed, paraffin-embedded tissue sections.

Anti-Vimentin antibody is intended for in vitro diagnostic use.

Specifications

Specificity:	Vimentin
Immunogen:	Vimentin from eye lens of pig
Clone:	V9
Isotype:	Mouse IgG1
Species reactivity:	Human +, mouse +, rat +, dog +, pig +, others not tested

Summary and Description

Vimentin is the most dominant intermediate filament of mesenchymal or mesenchymal-derived non-muscle cells. It is found i.e. in endothelial cells, all fibroblastic cells, macrophages, lymphocytes, melanocytes, and Sertoli cells. All lymphomas and sarcomas are expressing Vimentin.

In many epithelial cells and thereof derived carcinomas one found co-expression of Vimentin and cytokeratins.

Staining of Vimentin is especially utilised when characterising metastases of unknown origin.

Additionally, this Vimentin antibody of clone V9 is excellently suitable for checking the degree of fixation of formalin-fixed tissue samples (Battifora 1991).

Reagent provided

Mouse monoclonal antibody in TBS with carrier protein and preservative for stabilisation in the following formats:

Concentrate:	1 ml	(Cat. No. AIB-30182)
Concentrate:	0.5 ml	(Cat. No. AIB-30183)
Ready-to-use:	6 ml	(Cat. No. AIB-30181)

Dilution of primary antibody

Dilution of Nordic Biosite' concentrated antibody depends on the detection system used. The final working dilution must always be determined by the user. The elaboration of staining protocol should be done by an experienced specialist. For Nordic Biosite' recommendations see chapter 'Staining procedure'.








Storage and handling

The antibody should be stored at 2-8°C without further dilution.

Dilutions of the concentrated antibody should be done with a suitable antibody dilution buffer (e.g. BCB-20005/BCB-20006 from Nordic Biosite). The diluted antibody should be stored at 2-8°C after use. Stability of this working solution depends on various parameters and has to be confirmed by appropriate controls. The antibody provided is suitable for use until the expiry date indicated on the label, if stored at 2-8°C. Do not use product after the expiry date.

Positive and negative controls should be run simultaneously with all specimens. If unexpected staining is observed which cannot be explained by variations in laboratory procedures and a problem with the antibody is suspected, contact Nordic Biosite' technical support or your local distributor.

Explanations of the symbols on the product label:

	Catalog Number Bestellnummer Reference du catalogue		Batch Code Chargenbezeichnung Code du lot	 Manufacturer Nordic BioSite AB Propellervägen 4A S-183 62 Täby Sweden Tel: +46 (0)8 5444 33 40 Fax: +46 (0)8 756 94 90 info@nordicbiosite.com www.nordicbiosite.com
	Use By Verwendbar bis Utiliser jusqu'à		In Vitro Diagnostic Medical Device In vitro Diagnostikum Dispositif médical de diagnostic in vitro	
	Consult Instructions for use Gebrauchsanweisung beachten Consulter les instructions d'utilisation		Temperature Limitation Lagerungstemperatur Limites de température	

Precautions

Use through qualified personnel only.

Wear protective clothing to avoid contact of reagents and specimens with eye, skin and mucous membranes. If reagents or specimens come in contact with sensitive area, wash with large amounts of water.

Microbial contamination of the reagent must be avoided, since otherwise non-specific staining may occur.

ProClin300 and sodium azide (NaN₃), used for stabilisation, are not considered hazardous materials in the concentrations used. Reaction of sodium azide with lead or copper in drainage pipes can result in the formation of highly explosive metallic azides. Sodium azide should be discarded in a large volume of running water to avoid formation of deposits. Material safety data sheets (MSDS) are available upon request.

Staining procedure

Refer to the following table for conditions specifically recommended for this antibody. Also refer to detection system data sheets for guidance on specific staining protocols or other requirements.

Parameters

*Pre-treatment
*Control tissue
*Working dilution
*Incubation time

Nordic BioSites recommendations

Heat Induced Epitope Retrieval in Citrate Buffer pH 6.0 (BCB-20015/-20016)
Skin
1:100-1:200 (for concentrates)
30-60 minutes

Quality control

The recommended positive control tissue for this antibody is skin. We recommend carrying out a positive and a negative control with every staining run. Please refer to the instructions of the detection system for guidance on general quality control procedures.

Troubleshooting

If you observe unusual staining or other deviations from the expected results please read these instructions carefully, refer to the instructions of the detection system for relevant information or contact your local distributor.

Expected results

If you observe unusual staining or other deviations from the expected results please read these instructions carefully, refer to the instructions of the detection system for relevant information or contact your local distributor..

Limitations of the Procedure








Immunohistochemistry is a complex technique involving both histological and immunological detection methods. Tissue processing and handling prior to immunostaining, for example variations in fixation and embedding or the inherent nature of the tissue can cause inconsistent results (Nadji and Morales, 1983). Endogenous peroxidase, alkaline phosphatase or biotin may cause non-specific staining depending on the detection system used. Tissues containing Hepatitis B Surface Antigen (HBsAg) may give false positive results with HRP (horse radish peroxidase) detection systems (Omata et al, 1980). Inadequate counterstaining and mounting can influence the interpretation of the results.

Nordic Biosite warrants that the product will meet all requirements described from its shipping date until the expiry date is reached, if the product is stored and utilised as recommended. No additional guarantees can be given. Under no circumstances shall Nordic Biosite be liable for any damages arising out of the use of the reagent provided.

Performance characteristics

Nordic Biosite has conducted studies to evaluate the performance of the antibody for use with a standard detection system. The product has been found to be sensitive and specific to the antigen of interest with minimal or no cross-reactivity.

Explanations of the symbols on the product label:

 REF	Catalog Number Bestellnummer Reference du catalogue	 LOT	Batch Code Chargenbezeichnung Code du lot	 Manufacturer Nordic BioSite AB Propellervägen 4A S-183 62 Täby Sweden Tel: +46 (0)8 5444 33 40 Fax: +46 (0)8 756 94 90 info@nordicbiosite.com www.nordicbiosite.com
	Use By Verwendbar bis Utiliser jusqu'à	 IVD	In Vitro Diagnostic Medical Device In vitro Diagnostikum Dispositif médical de diagnostic in vitro	
	Consult Instructions for use Gebrauchsanweisung beachten Consulter les instructions d'utilisation		Temperature Limitation Lagerungstemperatur Limites de température	

Bibliography

Battifora H. Am J Clin Pathol 96:669-671, 1991

Azumi N und Battifora H. Am J Clin Pathol 88:286-296, 1987

Churg A et al. Am J Surg Pathol 9:360-365, 1985

Jasani B et al. Virchows Arch A Pathol Anat Histopathol 406: 441-448, 1985

Mullink H S et al. Virchows Arch B Cell Pathol Incl Mol Pathol 52:55-65, 1986

Nadji M and Morales AR Ann N.Y. Acad Sci 420:134-9, 1983








Omata M et al. Am J Clin Pathol 73(5): 626-32, 1980

October 15, 2013

Rev: A1013

Doc: DBE_AIB-30182/-30183/-30181

Explanations of the symbols on the product label:

	Catalog Number Bestellnummer Reference du catalogue		Batch Code Chargenbezeichnung Code du lot	 Manufacturer Nordic BioSite AB Propellervägen 4A S-183 62 Täby Sweden Tel: +46 (0)8 5444 33 40 Fax: +46 (0)8 756 94 90 info@nordicbiosite.com www.nordicbiosite.com
	Use By Verwendbar bis Utiliser jusqu'e		In Vitro Diagnostic Medical Device In vitro Diagnostikum Dispositif médical de diagnostic in vitro	
	Consult Instructions for use Gebrauchsanweisung beachten Consulter les instructions d'utilisation		Temperature Limitation Lagerungstemperatur Limites de température	