



## Rabbit anti-CD21

**Cat. No.: AIB-30195(0.5 ml Concentrate);  
AIB-30238 (6 ml Ready-to-use)**

### Instructions for use

#### Intended use

This antibody is designed for the specific localisation of CD21 protein in formalin-fixed, paraffin-embedded tissue sections and in frozen tissue sections. Anti-CD21 antibody is intended for in vitro diagnostic use.

#### Specifications

**Specificity:** CD21  
**Clone:** Ep3093  
**Isotype:** Rabbit IgG  
**Species reactivity:** Human +, others not tested

#### Summary and Description

CD21 is a transmembrane glycoprotein of molecular weight 140 kDa. The molecular weight of the soluble form is 130 kDa.

CD21 is expressed by follicular dendritic cells (FDC) and mature B-cells. On lymphoid tissue B-cells CD21 expression is strong in marginal zones and only moderate in mantle zone B-cells. B-cells in germ centres are mostly negative for CD21. The CD21 expression on B-cells in bone marrow is only weak or absent.

Additionally, CD21 was found on several types of epithelial cells and to a minor extent in T-cell acute lymphoblastic leukaemia cells, subpopulations of normal thymocytes, and mature T-cells.

CD21 is a useful marker for detection of structural rearrangements within the FDC (follicular dendritic cell) network often observed in malignant lymphoma.

#### Reagent provided

Rabbit monoclonal antibody from cell culture supernatant in buffer with carrier protein and preservative for stabilisation in the following formats:

**Concentrate:** 0.5 ml (Cat. No. AIB-30195)

**Ready-to-use:** 6 ml (Cat. No. AIB-30238)

#### Dilution of primary antibody

Dilution of Nordic Biosite' concentrated antibody depends on the detection system used. The final working dilution must always be determined by the user. The elaboration of staining protocol should be done by an experienced specialist. For Nordic Biosite' recommendations see chapter 'Staining procedure'.








#### Storage and handling

The antibody should be stored at 2-8°C without further dilution.

Dilutions of the concentrated antibody should be done in a suitable antibody dilution buffer (e.g. BCB-20005/BCB-20006 from Nordic Biosite). The diluted antibody should be stored at 2-8°C after use. The stability of this working solution depends on various parameters and has to be confirmed by appropriate controls.

The antibody provided is suitable for use until the expiry date indicated on the label, if stored at 2-8°C. Do not use product after the expiry date. Positive and negative controls should be run simultaneously with all specimens. If unexpected staining is observed which cannot be explained by variations in laboratory procedures and a problem

Explanations of the symbols on the product label:

|   |   |   |  |   |
|---|---|---|--|---|
|  | Catalog Number<br>Bestellnummer<br>Reference du catalogue   |  | Batch Code<br>Chargenbezeichnung<br>Code du lot  |  Manufacturer<br><br>Nordic BioSite AB<br>Propellervägen 4A<br>S-183 62 Täby<br>Sweden<br>Tel: +46 (0)8 5444 33 40<br>Fax: +46 (0)8 756 94 90<br>info@nordicbiosite.com<br>www.nordicbiosite.com |
|  | Use By<br>Verwendbar bis<br>Utiliser jusqu'à  |  | In Vitro Diagnostic Medical Device<br>In vitro Diagnostikum<br>Dispositif médical de diagnostic in vitro |   |
|  | Consult Instructions for use<br>Gebrauchsanweisung beachten<br>Consulter les instructions d'utilisation |  | Temperature Limitation<br>Lagerungstemperatur<br>Limites de température                                  |   |

with the antibody is suspected, contact Nordic Biosite' technical support or your local distributor.

### Precautions

Use through qualified personnel only.

Wear protective clothing to avoid contact of reagents and specimens with eye, skin and mucous membranes. If reagents or specimens come in contact with sensitive area, wash with large amounts of water.

Microbial contamination of the reagent must be avoided, since otherwise non-specific staining may occur.

Sodium azide (NaN<sub>3</sub>), used for stabilisation, is not considered hazardous material in the concentration used.

Reaction of sodium azide with lead or copper in drainage pipes can result in the formation of highly explosive metallic azides. Sodium azide should be discarded in a large volume of running water to avoid formation of deposits. A material safety data sheet (MSDS) for the pure substance is available upon request.

### Staining procedure

Refer to the following table for conditions specifically recommended for this antibody. Also refer to detection system data sheets for guidance on specific staining protocols or other requirements.

#### Parameters

\*Pre-treatment

20015/-20016)

\*Control tissue

\*Working dilution

\*Incubation time

#### Nordic BioSites recommendations

Heat Induced Epitope Retrieval (for example in Citrate Buffer pH 6.0 (BCB-

Tonsil, lymph node

1:50-1:200 (for concentrates)

30-60 minutes

### Quality control

The recommended positive control tissues for this antibody are tonsil and lymph node. We recommend carrying out a positive and a negative control with every staining run. Please refer to the instructions of the detection system for guidance on general quality control procedures.

### Troubleshooting

If you observe unusual staining or other deviations from the expected results please read these instructions carefully, refer to the instructions of the detection system for relevant information or contact your local distributor.

### Expected results

This antibody stains positive in the cytoplasmic membrane or in the cytoplasm in formalin-fixed, paraffin-embedded tissue sections. Further details about the expression pattern of CD21 can be found in the chapter 'Summary and Description'. Interpretation of the staining results is solely the responsibility of the user. Any experimental result should be confirmed by a medically established diagnostic procedure.

### Limitations of the Procedure








Immunohistochemistry is a complex technique involving both histological and immunological detection methods. Tissue processing and handling prior to immunostaining, for example variations in fixation and embedding or the inherent nature of the tissue can cause inconsistent results (Nadji and Morales, 1983). Endogenous peroxidase, alkaline phosphatase or biotin may cause non-specific staining depending on the detection system used. Tissues containing Hepatitis B Surface Antigen (HBsAg) may give false positive results with HRP (horse radish peroxidase) detection systems (Omata et al, 1980). Inadequate counterstaining and mounting can influence the interpretation of the results.

Nordic Biosite warrants that the product will meet all requirements described from its shipping date until the expiry date is reached, if the product is stored and utilised as recommended. No additional guarantees can be given. Under no circumstances shall Nordic Biosite be liable for any damages arising out of the use of the reagent provided.

### Performance characteristics

Nordic Biosite has conducted studies to evaluate the performance of the antibody for use with a standard detection system. The product has been found to be sensitive and specific to the antigen of interest with minimal or no cross-reactivity.

Explanations of the symbols on the product label:

|   |   |   |  |   |
|---|---|---|--|---|
|  REF | Catalog Number<br>Bestellnummer<br>Reference du catalogue   |  LOT | Batch Code<br>Chargenbezeichnung<br>Code du lot  |  Manufacturer<br><br>Nordic BioSite AB<br>Propellervägen 4A<br>S-183 62 Täby<br>Sweden<br>Tel: +46 (0)8 5444 33 40<br>Fax: +46 (0)8 756 94 90<br>info@nordicbiosite.com<br>www.nordicbiosite.com |
|      | Use By<br>Verwendbar bis<br>Utiliser jusque   |  IVD | In Vitro Diagnostic Medical Device<br>In vitro Diagnostikum<br>Dispositif médical de diagnostic in vitro |   |
|      | Consult Instructions for use<br>Gebrauchsanweisung beachten<br>Consulter les instructions d'utilisation |      | Temperature Limitation<br>Lagerungstemperatur<br>Limites de température                                  |   |

## Bibliography

Herrmann LM et al. Am J Pathol 162:1075-1081, 2003  
 Dillon KM et al. J Clin Pathol 55:791-794, 2002  
 Pileri SA et al. Histopathol 2002, 41:1-29, 2002  
 Kunihiro Maeda et al. J Histochem Cytochem 50:1475-1485, 2002








Biddle DA et al. Modern Pathol 15:50-58, 2002  
 Nadji M and Morales AR Ann N.Y. Acad Sci 420:134-9, 1983  
 Omata M et al. Am J Clin Pathol 73(5): 626-32, 1980

November 11, 2013

Rev: A1113

Doc: DBE\_AIB-30195/AIB-30238

Explanations of the symbols on the product label:

|   |   |   |  |   |
|---|---|---|--|---|
|  | Catalog Number<br>Bestellnummer<br>Reference du catalogue   |  | Batch Code<br>Chargenbezeichnung<br>Code du lot  |  Manufacturer<br><br>Nordic BioSite AB<br>Propellervägen 4A<br>S-183 62 Täby<br>Sweden<br>Tel: +46 (0)8 5444 33 40<br>Fax: +46 (0)8 756 94 90<br>info@nordicbiosite.com<br>www.nordicbiosite.com |
|  | Use By<br>Verwendbar bis<br>Utiliser jusqu'à  |  | In Vitro Diagnostic Medical Device<br>In vitro Diagnostikum<br>Dispositif médical de diagnostic in vitro |   |
|  | Consult Instructions for use<br>Gebrauchsanweisung beachten<br>Consulter les instructions d'utilisation |  | Temperature Limitation<br>Lagerungstemperatur<br>Limites de température                                  |   |